# **Tropsensor: A Rapid Non-Invasive Transdermal Solution For Troponin Measurement**

Jitto Titus<sup>1</sup>, Alan HB Wu<sup>2</sup>, Siddharth Biswal<sup>1</sup>, Atandra Burman<sup>1</sup> and Allan S Jaffe<sup>3</sup>

### BACKGROUND AND OBJECTIVE

Over 10 million patients present at the ED with chestpain, with the majority due to non-cardiac causes, resulting in unnecessary burdens for Emergency Departments (ED)1

Current biochemical biomarker testing relies on labeling the cTn with turnaround times often an hour or more<sup>2</sup>.

Point-of-care solutions reduce the test time, but still depend on the proper handling of samples and blood processing techniques that could lead to difficulties<sup>3</sup>



Solution: Tropsensor<sup>4,5</sup> provides a rapid (5-min) non-invasive alternative to standard of care (SOC) hs-cTn measurements without the need for a blood draw.

## METHODS AND MATERIALS

Study site: Zuckerberg San Francisco General

Cohort size: 26 patients

Recruitment mechanism: Patients presenting with ACS-related symptoms at the ED

Technology tested: Tropsensor, a wrist-worn vibrational spectroscopy-based, noninvasive transdermal troponin sensor

Comparative technology: Siemens ADVIA Centaur<sup>6</sup> high sensitivity troponin-I assay via a blood draw




## SUMMARY

A three-layer neural network was used to train regression and binomial classification models.

This neural network contains three 1-D convolutional layers stacked with batch normalization and Rectified Linear Unit (ReLU)7 layers.

The neural network is trained using Adaptive Movement (ADAM)<sup>8</sup> stochastic optimizer employing a cross entropy loss function.

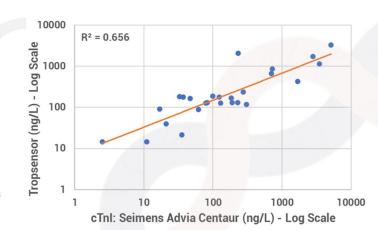


The 99th percentile6 cutoff for the SOC Seimens Advia Centaur assay is 47.34 ng/L

#### CONCLUSION AND DISCUSSION

Tropsensor output correlates significantly with SOC hs-cTn assay

- · accelerated assessment of patients presenting with chest pain
- · since blood draw is not required, Tropsensor could provide a rapid, safe, standardized and reliable source for cTn while allowing bedside serial trending.
- · potential for streamlining cardiac care workflow by ruling-out many non-cardiac patients and identifying those with high values who are at

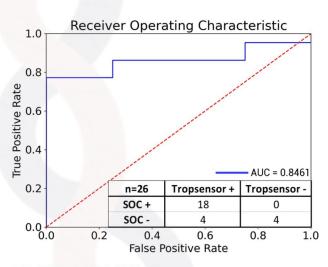

Sensitivity = 1, Specificity = 0.5, PPV = 0.8182, NPV = 1, Correlation = 0.8228, Accuracy = 0.8461

Musetti, V et al. Evaluation of the analytical performance of a new ADVIA immunoassay using the Centaur XPT platform system for the measurement of cardiac troponin I. Clin. Chem. Lab. Med. (CCLM) 56, e229–e231 (2018) SI, Jiong, Sarah L. Harris, and Evangelos Yfantis. \*A dynamic ReLU on neural network. \*2018 IEEE 13th Dallas Circuits and Systems Conference (DOS), IEEE, 2018.

Zhang, Zijun. 'Improved adam optimizer for deep neural networks. \*2018 IEEE/ACM 26th International Symptom on Quality Service (IWOS), IEEE, 2018.





